



perfect scan = perfect fit

Rejected Scans - Need to Rescan Patient

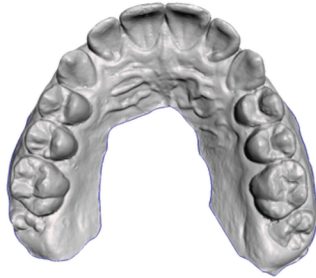

Lacks the complete palate and would not work for any acrylic appliance. **Cannot be fixed digitally.**


This would be acceptable for Essix ONLY

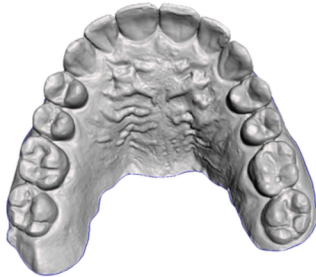

Lacks palate definition in the posterior region required up to the distal of the 2nd molars.


This scan would require additional fees to fix/repair the scan. In periods of high volume, this scan would be rejected.

Potentially Acceptable - May Incur Fees

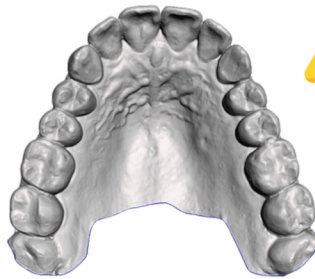

Contains enough palatal definition back into the area of the 2nd molars. However, the **lack of a complete palatal scan may require additional fees** depending on appliance design.

Ideal and Absolutely Perfect Scans

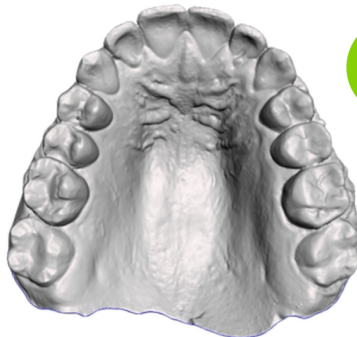

Perfect Scan to make ANY appliance. All teeth appear to be scanned completely without any distortions. All palatal areas are scanned and cover the complete area between the distal most teeth.